

9L Wall Bracket with Security Cable Installation Instructions



What's included

- ✓ **9L Secure a Drug Wall Bracket**
Product code: 2401300000
- ✓ **Blue Security Cable**

What's not included

- ✗ **4x Toggle Bolts or Plastic Conical Anchors**
(recommended to best accommodate the weight of full containers)
 - **Toggle bolt** used for Drywall
 - **Plastic Conical Anchor + Screws** used for Concrete Tile/Masonry
- ✗ **4x Washers**
- ✗ **Padlock**

Mounting

The recommended mounting height is 43" from the top of the bracket to the ground. To allow space for the containers, ensure there are no objects within 12" above the top of the bracket.

Installation:

1. Place the bracket on the wall in the desired location. The recommended mounting height is 43" from the ground. Make sure it is leveled.
2. Mark your 4 holes prior to drilling.
3. Drill holes through the 4 mounting holes.
4. Mount the bracket in position using your toggle bolts or anchors.
5. Place the 9L Secure a Drug bottle inside the wall mount.
6. Thread the blue security cable through the handle of the side of the wall mount and the handle of the Secure a Drug bottle.
7. Attach the ends of the security cable with a padlock.
8. Ensure the keys to the padlock are stored in a secure location and the appropriate staff members have access to the key.

SAD251010

