




SECURE a DRUG

The simple
solution

for the safe disposal
of controlled substances

Secure a Drug is the easy and responsible way to dispose of controlled substances. Our solution neutralizes pharmaceuticals in just three steps, keeping your patients and fellow clinicians safe.

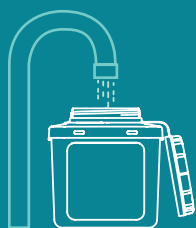

Fill with hot water
to the fill line



Dispose of medications,
close container and
gently shake




When contents
reach the fill line,
close and lock

While regulatory compliance is critical, the responsibility we have as medical professionals to care for and keep patients and fellow clinicians safe is what drives us.

Secure a Drug is the simple solution for deactivating and neutralizing pharmaceutical controlled substances. Easy to use, Secure a Drug allows for the permanent removal of potential threats and abuse associated with the opioid crisis and beyond, reducing risk to those most vulnerable.



SCAN HERE
TO LEARN MORE

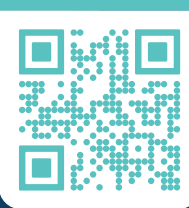

866 381 8889



WWW.SECUREADRUG.COM



PRODUCT SPECIFICATIONS

Each Secure a Drug container is supplied with Activated Carbon that meets the following specifications:

- 1230 Iodine, metallurgical grade bituminous coal-based activated carbon
- Granular in appearance - 12US MESH, 1.7mm to 40 US MESH 0.425mm
- 2L product will neutralize 500 pills and 4L product will neutralize 1000 pills

The 44mm diameter opening with funnel shaped surround make it easy to pour pills and liquids into the container.

Our purpose-designed wall-mountable locking bracket is compatible with both sizes and enables agitation without time spent unlocking and removing the container.

NOT SUITABLE FOR DRUGS THAT ARE SUPPLIED WITH SPECIFIC DISPOSAL INSTRUCTIONS. WHEN USED IN HEALTHCARE FACILITIES, NOT FOR USE WITH RCRA LISTED DRUGS OR EFFERVESCENT MEDICATIONS.

